

Insulin
U-100
1.0 mL

English




UNI TRACT  [®]TM
Passive Sharps Protection

1mL Insulin Syringes with retracting needle

Quick Reference Guide


Instructions for Cartridge or Ampoule Use

- 1** A. Remove cap.
B. Push down on plunger to expel excess air.ⁱ

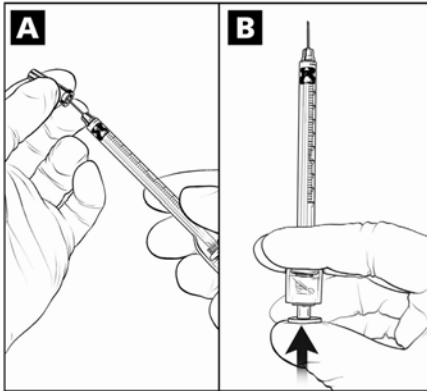


I. Patients who mix two types of insulin should not change the order or process of mixing.

- 2** A. Draw up required dose—always pull the plunger beyond the 20 unit mark.ⁱⁱ
B. Expel any excess air or fluid.ⁱⁱⁱ

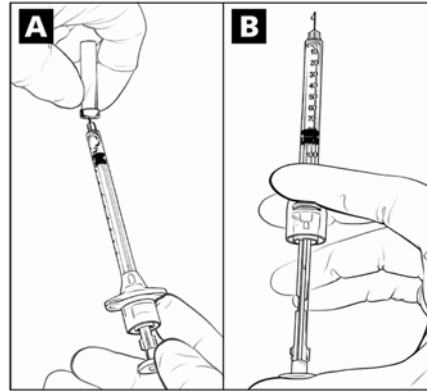


- II. To initiate injection—for doses less than 20 units, ensure the plunger is pulled back beyond the 20 unit mark.
III. Prepare and give injections using aseptic technique according to institutional policy, or as directed by a healthcare professional.

- 3** Insert needle, then push down on the plunger to inject entire dose, passing through any resistance from the needle engagement.^{iv}

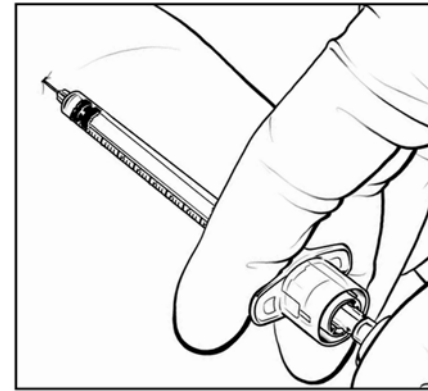

- IV. Operators should feel and hear the activation of the automatic retraction mechanism. Pressure should be applied on the plunger until it reaches the zero mark.

- 4** Operators can control the speed of automatic needle retraction^v directly from the body.^{vi}

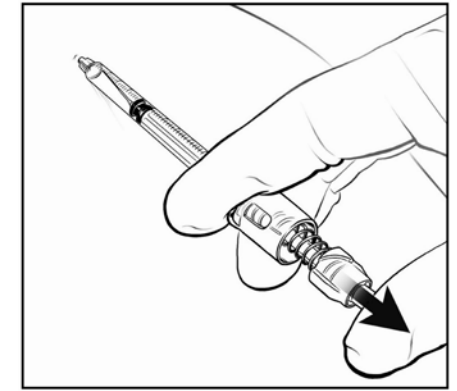

- V. By adjusting thumb and finger pressure on the plunger.
VI. After use, discard in an appropriate container.

