

Tuberculin
1.0 mL

English




UNI TRACT  [®]TM
Passive Sharps Protection

1mL Tuberculin Syringes with retracting needle


Quick Reference Guide

Instructions for Cartridge or Ampoule Use

- 1** A. Remove cap.
B. Push down on plunger to expel excess air.

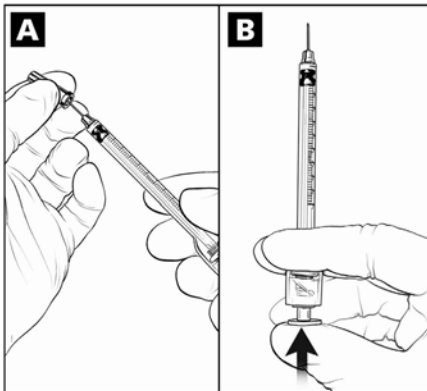


- 2** A. Draw up required dose—always pull the plunger beyond the 0.2mL mark.^I
B. Expel any excess air or fluid.^{II}

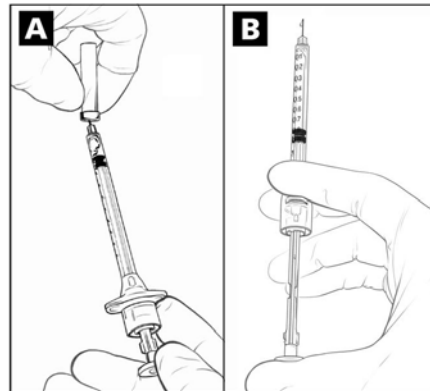


- I. To initiate injection—for doses less than 0.2mL, ensure the plunger is pulled back beyond the 0.2mL mark.
II. Prepare and give injections using aseptic technique according to institutional policy, or as directed by a healthcare professional.

- 3** Insert needle, then push down on the plunger to inject entire dose, passing through any resistance from the needle engagement.^{III}

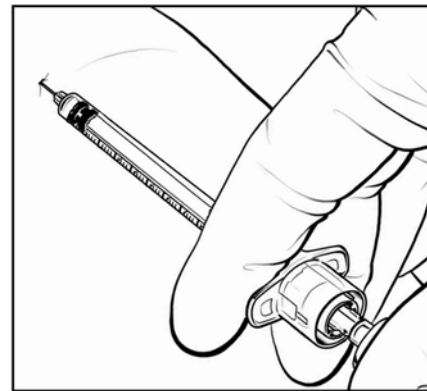

- III. Operators should feel and hear the activation of the automatic retraction mechanism. Pressure should be applied on the plunger until it reaches the zero mark.

- 4** Operators can control the speed of automatic needle retraction^{IV} directly from the body.^V

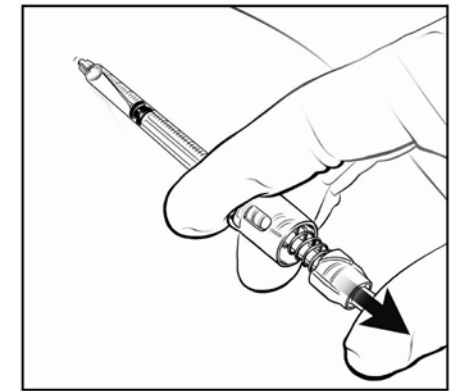

- IV. By adjusting thumb and finger pressure on the plunger.
V. After use, discard in an appropriate container.

