

Safe
1.0 mL

English




UNI TRACT  [®]TM
Passive Sharps Protection

1mL Safe Syringes with retracting needle


Quick Reference Guide Instructions for Use

- 1** A. Remove cap.^I
B. Push down on plunger to expel excess air.

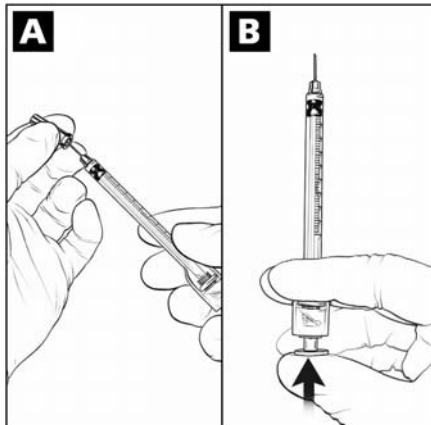


I. Prepare and give injections using aseptic technique, as directed by a healthcare professional or needle exchange program.

- 2** A. Draw up required dose.
B. Pull the plunger beyond the 0.8mL mark.^{II}

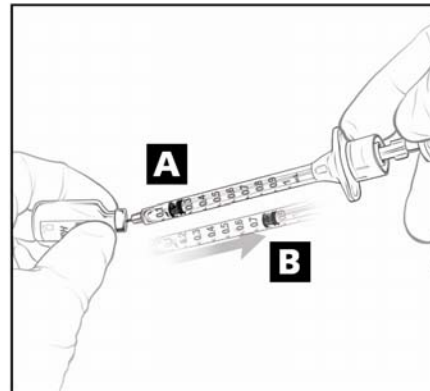

II. To initiate injection—for doses less than 0.8mL, ensure the plunger is pulled back beyond the 0.8mL mark.


- 3** A. Expel excess air or fluid.
B. Insert needle, then push down on the plunger to inject entire dose,^{III} passing through any resistance from the needle engagement.^{IV}

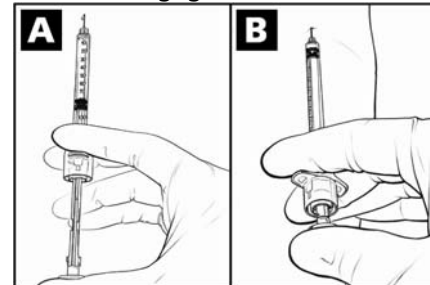

III. This syringe has a gated reuse prevention feature; once the plunger is depressed beyond the 0.5mL mark, the syringe cannot be reloaded, although aspiration and blood witness is permitted between each gate.

IV. Operators should feel and hear the activation of the automatic retraction mechanism. Pressure should be applied on the plunger until it reaches the zero mark.

- 4** A. Operators can control the speed of automatic needle retraction directly from the body.^V
B. Dispose of the syringe safely.^{VI}

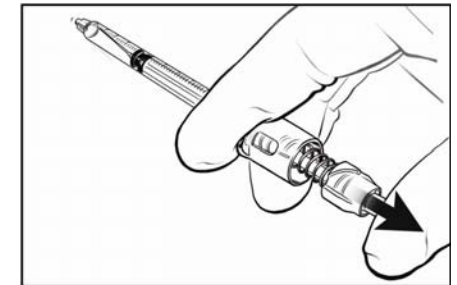

V. By adjusting thumb and finger pressure on the plunger.
VI. After use, discard in an approved sharps container or return to a Needle Exchange facility.

UNILIFE  [™]
medical solutions

Unilife Medical Solutions is an ISO 13485 certified developer and supplier of innovative safety devices.

For the entire instructions for use, please visit www.unitract.com

10-9851 Rev. 01